

fix & sauber
easy & clean

Schuhüberzug - Box / shoe cover machine

im praktischen Kofferformat

Einmal-Schuhüberzüge bequem und hygienisch anziehen. Keine Berührung mit dem Kontaminationsbereich. Mechanische Funktionsweise, kein Stromanschluss nötig, daher überall einsatzbereit. Ideal für Bereiche mit hohem Hygieneanspruch wie Krankenhäuser, Reinräume, Labore, Lebensmittelverarbeitung usw.

Einfachste Anwendung:
Fuß mit leichtem Druck in den Spender stellen, Schuhüberzug legt sich automatisch über den Fuß.

Abmessung: ca. 47 x 25 x 19 cm

Gewicht: 5,85 kg

Kapazität: 50 Stück

Gehäuse: Kunststoff, grau mit Tragegriff



SCHUHÜBERZUG - BOX, LEER

Art.-Nr. 04 890

NACHFÜLLPACK SCHUHÜBERZÜGE

Art.-Nr. 04 891

Befüllen der Schuhüberzug - Box

- 1** Box öffnen, Balken nach unten drücken bis sie einrasten.



- 2** Ringe der Überziehschuhe an den 4 Eckposten befestigen und nach unten drücken.



- 3** Den Draht losbinden.



- 4** Entriegelungstasten leicht drücken - Balken schiebt sich nach oben, Deckel schließen. Gebrauchsfertig.



Es können ca. 50 Stück Schuhüberzüge geladen werden.



Sänger GmbH • Industriegebiet Herdwiesen • 74575 Schrozberg
Tel: 0 79 35 / 72 24 0 • Fax: 0 79 35 / 72 24 199
info@sanger.de • www.sanger.de

fix & sauber
easy & clean

Schuhüberzug - Box / shoe cover machine

Ideal shoe cover machine for comfortable, speedy and hygienic donning of shoe covers. No need of touching the contaminated area. Mechanically functioning machine does not require electricity and can be used anywhere. Portable case is ideal to be used in areas of high requirement of hygienic conditions, i.e. hospitals, clean rooms, laboratories, places of food processing.

User instructions:

Place the foot and press lightly.

Automatically one shoe cover will hug your shoe.

Remove your foot and repeat the same motion with other foot.



Size: approx. 47 x 25 x 19 cm

Weight: 5,85 kg

Capacity: 50 Stück

Gehäuse: Kunststoff, grau mit Tragegriff


SHOE COVER BOX, EMPTY

Art.-No. 04 890

REFILL PACK SHOE COVER BOX

Art.-No. 04 891

- 1** Open the case, press metal bar till it becomes engaged.

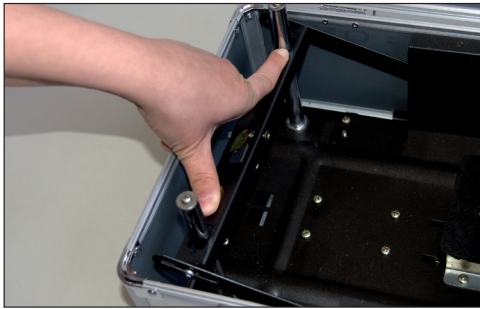

- 2** Rubber bands of the shoe covers to be placed on 4 edges, pressing lightly.



- 3** Remove the wire.



- 4** Lightly press the locking knob, the metal bar will slide upwards. Close lid. Machine is ready for use.



Up to about 50 shoe covers can be placed in the machine.



Säger GmbH • Industriegebiet Herdwiesen • 74575 Schrozberg/Germany
Tel: 0049 (0) 79 35 / 72 24 0 • Fax: 0049 (0) 79 35 / 72 24 199
info@sanger.de • www.sanger.de